



TISSUE SHIPPING INSTRUCTIONS / LIST OF CONTENTS

Tissue Kit Shipping Instructions:

Please Note: **Sample containers must be labeled with the patient's name.** Multiple samples can be sent in one kit as long as each separate container is properly labeled with patient's name. Please consult the List of Contents below to ensure all of the kit materials have been provided. You can download a copy of this form for your own records at: www.frylabs.com/forms.php. This kit is manufactured by TheraPack and is IATA, DOT, and USPS compliant for "Category B UN3373".

1. Check the expiration date listed on the exterior box of the collection kit.
2. Freeze the cool pack that is included with the kit.
3. Tissue samples should **NOT** contain or be stored in any preservatives or solutions. Sample(s) should be placed in the provided sterile container and shipped using these instructions.
4. Place the cool pack, sample container(s), and the enclosed absorbent paper inside of the styrofoam box.
5. Use the included red tape to seal the styrofoam box by taping around the seam between the styrofoam box lid and base.
6. Place the styrofoam box inside of the enclosed biohazard sample bag. The completed requisition (**it must be filled out completely or delay of testing may occur**) should be placed in the back pocket of this bag, **NOT** inside the box with the sample.
7. Place the styrofoam box inside of the provided white cardboard box along with this sheet and close the box (copies of these instructions are available on our website at www.frylabs.com/forms.php). If a UN3373 label is not already on the white box please adhere it on the bottom of the box.
8. Place the box inside the FedEx shipping bag and seal the bag shut. Note: Please review at the billable stamp on the front of the enclosed clinical pack. If it says *2-Day*, ship your sample **Monday, Tuesday, or Wednesday**. If it says *Standard Overnight*, ship your sample **Monday, Tuesday, Wednesday, or Thursday**.
9. Take the package to your nearest FedEx location and they will help you from there. Go to <http://fedex.com/us/> or call 1.800.463.3339 to determine the nearest FedEx location or for assistance with your shipping needs.

List of Contents:

This item list must be placed between the secondary receptacle(s) and outer packaging in accordance with IATA Packing Instruction 650.

<input checked="" type="checkbox"/> Diagnostic Specimen Containers	<input checked="" type="checkbox"/> Absorbent Pad(s)
<input checked="" type="checkbox"/> Requisition	<input checked="" type="checkbox"/> Instructions/List of Contents
<input checked="" type="checkbox"/> Styrofoam Container	<input checked="" type="checkbox"/> Biohazard Bag
<input checked="" type="checkbox"/> Cool Pack/Refrigerant	<input checked="" type="checkbox"/> Waterproof Tape