



## TISSUE SHIPPING INSTRUCTIONS / LIST OF CONTENTS

### **Tissue Kit Shipping Instructions:**

Please Note: **Sample containers must be labeled with the patient's name to process the sample.** Multiple samples can be sent in one kit if each separate container is properly labeled with patient's name. Please consult the List of Contents below to ensure all of the kit materials have been provided. You can download a copy of this form for your own records at: [www.frylabs.com/forms](http://www.frylabs.com/forms) This kit is IATA, DOT, and USPS compliant for "Category B UN3373".

1. Inspect the kit to ensure it is in good condition and has not been damaged during shipment or storage. The sample collection containers should be sealed.
2. Freeze the cool pack that is included with the kit.
3. Legibly label the plastic sample cup with the patient name, collection date, and collection time. Tissue samples should **NOT** contain or be stored in any preservatives or solutions. Sample(s) should be placed in the provided sterile container and shipped using these instructions.
4. **Do not use during surgical procedures. Only sample from materials already removed from the patient in an area outside of the procedure room.** Sample(s) should be placed in the provided sterile container and shipped using these instructions. Aseptic collection technique is very important during collection and sampling.
5. Place the sample container(s) and the enclosed absorbent paper inside of the enclosed biohazard sample bag. The completed requisition (**it must be filled out completely or delay of testing may occur**) should be placed in the back pocket of this bag, **NOT** inside of the bag with the sample containers.
6. Place the cool pack and the assembled biohazard bag inside of the styrofoam box. Use the included red tape to seal the styrofoam box by taping around the seam between the styrofoam box lid and base.
7. Place the styrofoam box inside of the provided white cardboard box along with this sheet and close the box (copies of these instructions are available on our website at [www.frylabs.com/forms](http://www.frylabs.com/forms). If a UN3373 label is not already on the white box please adhere it to the box.
8. Place the box inside the FedEx shipping bag and seal the bag shut. Note: Please review at the billable stamp on the front of the enclosed clinical pack. If it says *2-Day*, ship your sample **Monday, Tuesday, or Wednesday**. If it says *Standard Overnight*, ship your sample **Monday, Tuesday, Wednesday, or Thursday**.
9. Take the package to your nearest FedEx location and they will help you from there. Go to <http://fedex.com/us/> or call 1.800.463.3339 to determine the nearest FedEx location or for assistance with your shipping needs.

### **List of Contents:**

This item list must be placed between the secondary receptacle(s) and outer packaging in accordance with IATA Packing Instruction 650.

Diagnostic Specimen Containers  
 Requisition  
 Styrofoam Container  
 Cool Pack/Refrigerant

Absorbent Pad(s)  
 Instructions/List of Contents  
 Biohazard Bag  
 Waterproof Tape